


# Quick Start Guide

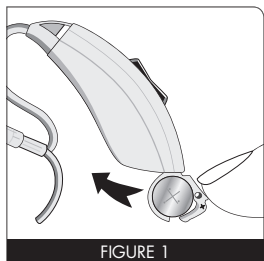


## 1. Prepare for use

Attach either a left or right tube.  
Insert battery and close the battery door.  
Your TuneAmp is now on. (Figure 1)

## 2. Place TuneAmp over your ear.

Position the tip in the ear canal and the retaining lock in the bowl of your ear.

(Figure 2)

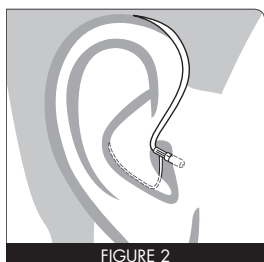


## 3. Try out different amplification settings


Start out in a quiet environment. When you first turn TuneAmp on, it will be at the lowest level of amplification. How does it sound?

There are four different amplification settings for you to try. Press and release the upper rocker switch. (Figure 3)

You will hear two (2) beeps, indicating you are now at the next setting, and so on, up to four levels of amplification.

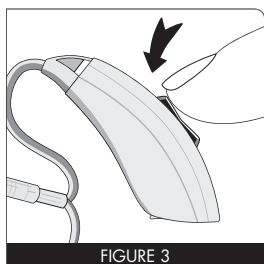

Continue sampling the settings until you decide which one you like best.

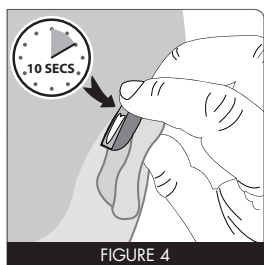

## 4. Lock in your preferred setting

Press and HOLD the rocker switch for ten (10) seconds, until you hear a long beep.

(Figure 4)

Congratulations! Your set-up is complete. TuneAmp remembers your preferred amplification setting from now on.

**TuneAmp FOCUS:** You now have two programs available – Quiet and Noise. To switch between Quiet and Noise programs, **press and hold** the rocker switch for **TWO (2)** seconds. You will hear a confirmation tone: a single beep for Quiet, “Shhh” for Noise. In either program, you can still adjust the volume by quickly **tapping** the rocker switch.

**TuneAmp BASIC:** You have one program available – Quiet. You can still adjust the volume by **tapping** the rocker switch.

**Important:** To extend battery life, turn your TuneAmp off by opening the battery door whenever the device is not in use.